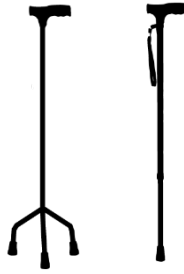
GR

EN

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MEDICAL • CASES • FIREFIGHTING est. 1977

Intended Use:

The canes are walking aids that can be used with one hand and are aimed for people with limited mobility and difficulty for walking, for disabled or patients to help move in and out of the house, or/and their rehabilitation.

Assembling:

The Canes are delivered assembled & ready to use.

To adjust the height, press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished.

Before use:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check the correct assembly of the device.
3. Check if the safety washer is tightened well.

Maintenance & Cleaning:

- Clean each section of the walker frequently.
- Use only a cloth and mild soap on surfaces that need cleaning.
- Then use a clean cloth and dry well before using the product.
- If you need to disinfect the product, use a common, mild disinfectant.

CAUTION (!):

In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.

Further Instructions:

Hold the cane with the opposite hand to the weak leg and with the longer side of the support base directed toward the inside, if it is 3-leg or 4-leg cane. Bring the product forward together with the foot of the week leg loading the full weight on the stronger leg.

During walking, be careful not to place the device neither too close to the body to avoid tripping over one of tips nor too far away to avoid an imbalance. In case of use on stairs, verify that the size of the step are sufficient to contain the base of device.

To go up the stairs, the user have to put first the cane on the stair, second to raise the stronger leg, and last the weaker leg. To go down the stairs with the cane must follow the injured leg, and finally the good leg. If necessary, get help from an assistant.

Environmental Protection:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

Declaration of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides

Warranty:

The product is guaranteed for a period of one (1) year from the purchasing date.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

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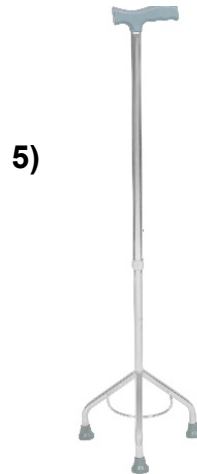
Description of symbols that have been printed on the label, Master Carton and or User Manual

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MOBIAC S.A