GR EN

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MOBIAK S.A
KATHIANA AKROTIRIOU
CHANIA, CRETE-GREECE
T: +30 28210 63222

Intended Use:

The walkers can be used by people with limited mobility for travel within and outside the home.

Assembling:

The walkers are delivered assembled & ready to use.

Further Instructions:

1. To fold or unfold the walker, use the handles or the button (if available)
2. When using the walker for sitting or standing up push the handles down, securing that the walker will stand still and not slip.
3. When you get out of the walker must rely on both handles equally.
Leaning only to one handle (without escort) can damage the handle.

Storage & Transportation:

Close and fold the walker using the handles or button before storing or transporting it

Maintenance & Cleaning:

- Clean each section of the walker frequently.
- Use only a cloth and mild soap on surfaces that need cleaning.
- Then use a clean cloth and dry well before using the product.
- If you need to disinfect the product, use a common, mild disinfectant.

CAUTION (!):

In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.

IMPORTANT (!)

Before using the walker **MAKE SURE** that :

- The walker is in full extension and is locked in this position .
- The wheels (if available) are firmly fixed
- The handles are the same height and the screws that secure them on the frame are tight.
- There are no cracks or fractures in the frame.
- Regularly clean the walker.
- Do not let the walker wet.
- Do not let children near the walker without supervision

Environmental Protection:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover the parts that can wear out during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

Warranty Claim Form

Buyers Data

Name:

Date of purchase:S/N.....

LOT:.....Phone:

Resellers Data

Name :
.....
.....

Address:.....
.....

Phone:

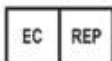
Reseller Stamp

Valentine International Ltd.
8th Floor, No. 149, Sec 2, Ta Tung Road
221 Hsichih District, New Taipei City
Taiwan R.O.C.

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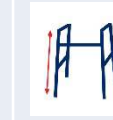
1)

Valentine International Ltd.
8th Floor, No. 149, Sec. 2,
Ta Tung Road, 221 Hsichih District,
New Taipei City, Taiwan R.O.C.
Tel: +886 2 8692 6662
Fax: +886 2 8692 6660
info@valentine-dynalife.com

Declaration of Conformity

Manufacturer: Valentine International Ltd.
221 Hsichih District, New Taipei City
Taiwan R.O.C.

Authorized Representative: Y. Sung Handelsvertretung
Toulouser Allee 9 40211 Dusseldorf
Deutschland


Products:

Valentine Item No.	Description
10184-Bariatric	Bariatric Folding Walker
10297/T/BAG	Roll Up Wheelchair Ramp with Bag
10298LH	Portable Wheelchair Ramp
10563	Fracture Bedpan with Lid
11613/16/WH	Suction Grab Bars
12312	Fold Away Grab Bar
14437	Heel/Elbow Protector
60781/R to 60789/R	Dyna-Seal Cast & Bandage Protectors
61016	Deluxe Inflatable Shampoo Ring

I hereby declare that the products specified above are classified as class 1 devices, as in conformity with the requirements of the EU Medical Device Regulation (EU) MDR 2017/745.

This was verified with conformity evaluation procedures according to Medical Device Regulation Annex VIII.

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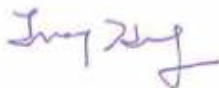Description of symbols that have been printed on the label, Master Carton and or User Manual

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