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INTENDED USAGE:

The Shower Seat allows the user an easy transfer to the shower or bathtub and an increase in independence during the shower stage of daily personal hygiene, reducing muscle fatigue due to standing position as well as slip and injuries risks. The non-slip rubber feet ensure grip with the floor, and the versions provided with the height adjustable legs allow the user to easily change the total height according to their requirements. The backrest, gives additional safety and comfort. The shower seat can be easily moved due to its light weight.

INDICATIONS AND CONTRAINDICATIONS:

The Shower Seat is aimed to elderly people, wheelchair users, people with reduced mobility of lumbar, knee or hip, and to all those who have difficulty to maintain the standing position during the shower stages of daily personal hygiene. The user is required to have autonomy of balance during the sitting and standing procedures as well as to have adequate ability to keep sitting. The shower seat must not be used if the user has problems related to balance or perception disorders.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- If you don't read this user manual, it is preferable not to use this product or another available part.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts
- Protect the product from scratches, cuts and punctures.

SAFETY WARNINGS AND INDICATIONS:

- The maximum user weight indicated in the instructions must not be exceeded.
- Before every use, be sure the seat is correctly assembled in a stable position.
- The seat can be used only on surface that are free from obstacles.
- The risk of finger injuries may increase during the mounting or assembling process.
- Do not swing on the seat, this may lead to slipping, overturning or subsidence of the legs. If you are using the shower chair, always keep the seat on its 4 legs.
- Do not modify the seat and its components in any way.
- Do not use the shower seat if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- After use, the shower seat should be removed from the shower tray to avoid any obstruction.
- Do not use the shower seat for a different purpose from the intended one.
- The shower seat must not be stored outdoors.

MAINTENANCE INFO:

In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.
5. If you need to disinfect the product, use a common, mild disinfectant

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the products' components.

INCIDENT REPORTING:

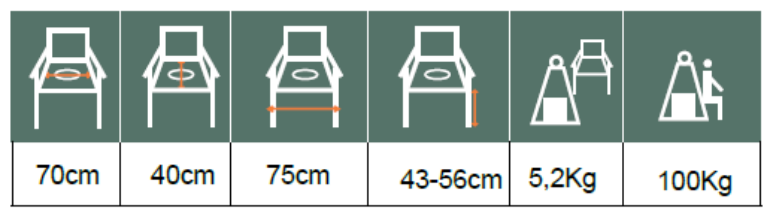
If there was a serious incident involving this product, contact the manufacturer of this product at mobiakcarequality@mobiak.com

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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Assembling:

The product is delivered assembled & ready to use. You may need to assemble the seat with the backrest. The product that you have in your hands may differ from the one in the following photos. The assembly process is similar.

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To install the Backrest onto the seat, insert at the same time the tubes of the backrest into the special slots under the seat until they lock in place.

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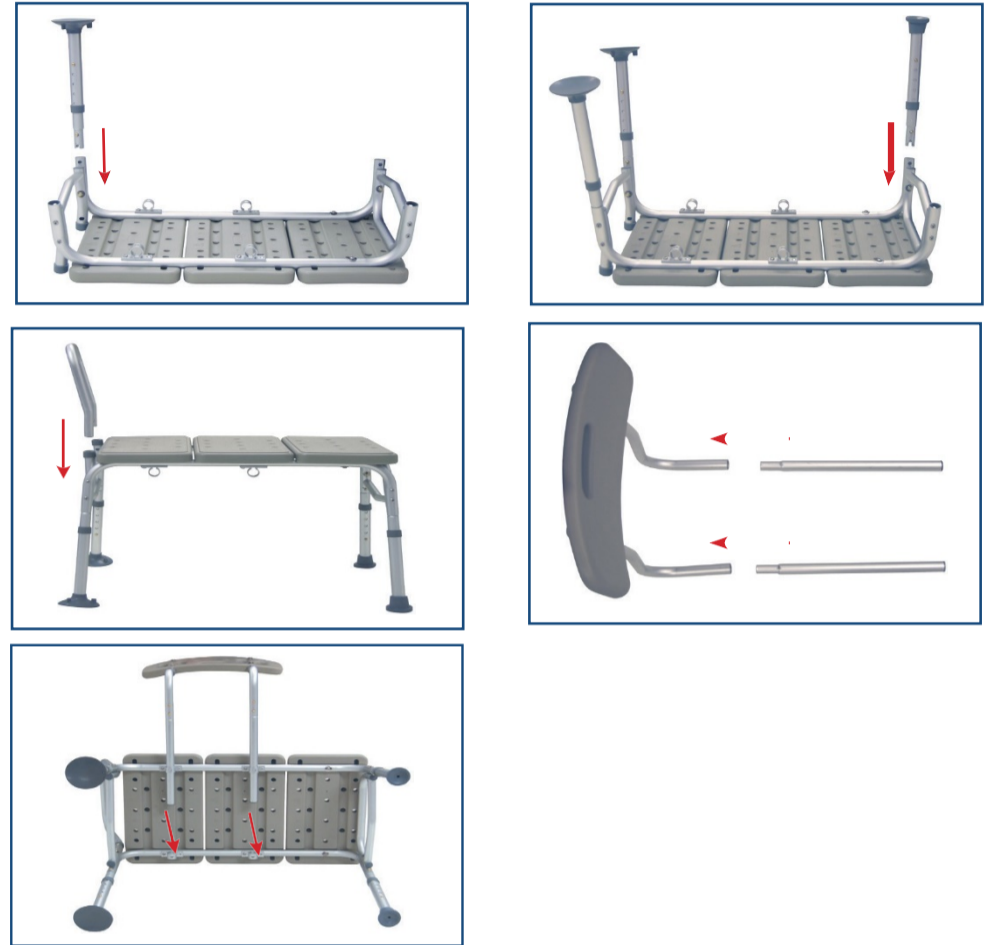
WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date. The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT cover the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer. No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

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USER MANUAL: BATHTUB TRANSFER CHAIR

REF :0810067

MOBIAK S.A
ΚΑΘΙΑΝΑ ΑΚΡΟΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiacare.com