0804900_0808038_0804902

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0804902 TALOS - TiMOTION actuator with payload 180 Kg

2. INTRODUCTION

Thank you for choosing a patient lift/stand up lift from the **PEGASUS - ATLAS - TALOS by MOBIAK S.A** product range by MOBIAK S.A. Their design and quality are a guarantee of comfort, safety and reliability. **PEGASUS - ATLAS - TALOS by MOBIAK S.A** patient lifts/stand up lifts by MOBIAK S.A are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the lift you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the patient lift/stand up lift. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

PEGASUS - ATLAS - TALOS by MOBIAK S.A patient lifts/stand up lifts are intended to lift and move people with motor difficulties. An assistant operates the machine. The lifts cannot be used by only one person. MOBIAK S.A refuses all liability for damages caused by improper use of the device or by its use for a purpose other than that one specified in this manual.

4. DECLARATION OF EC CONFORMITY

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

5. GENERAL WARNINGS

WARNING!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual, checking with a particular attention at the warnings before use.
- For proper use of the device to consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongsto.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instructions

Medical Device

Conditions of disposal

Attention should read the instructions.

Production dates

Waste Electrical & Electronic Equipment (WEEE)

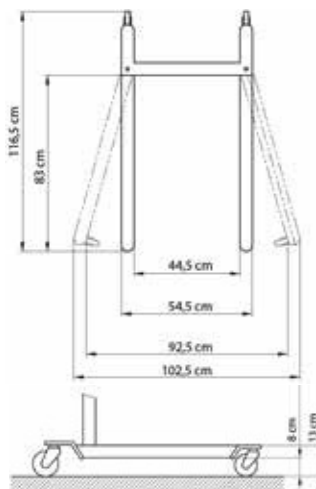
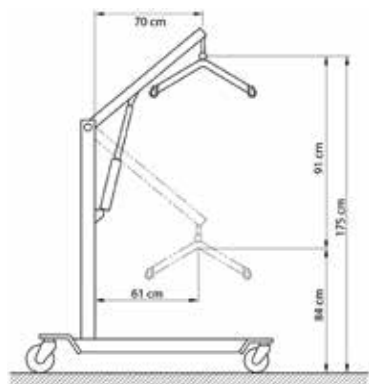
Protection against dust and water splashes

B Type

7. DIMENSIONS



7.1 Dimensions of structure with payload 135 Kg

0804900 PEGASUS


7.2 Dimensions of structure with payload 150/180 Kg

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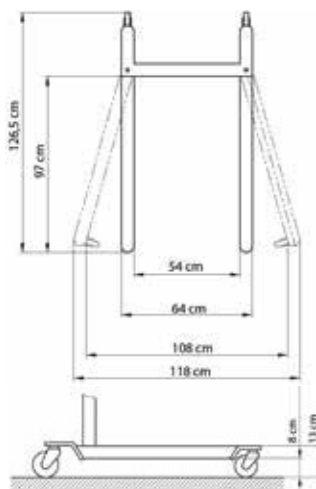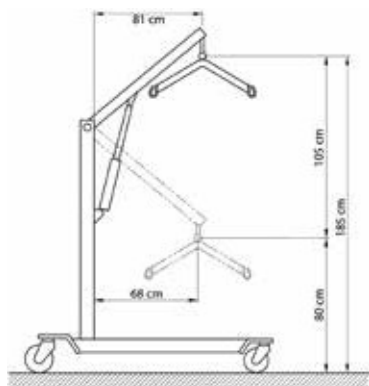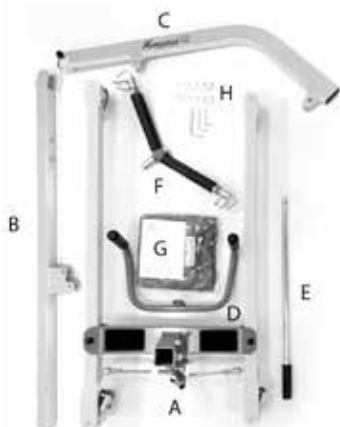
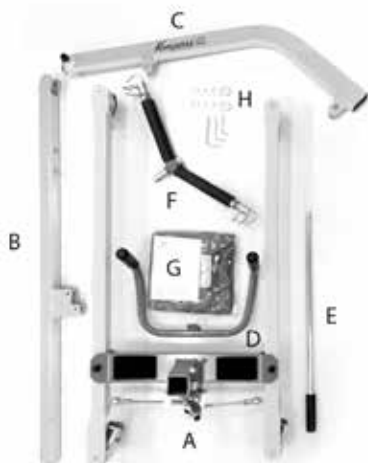
9 DESCRIPTION OF THE ACTIVATION KITS

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




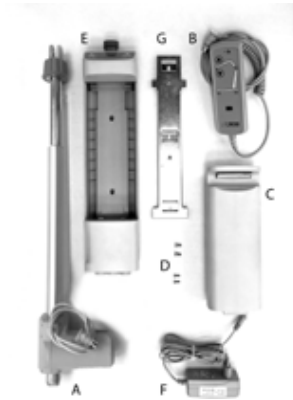
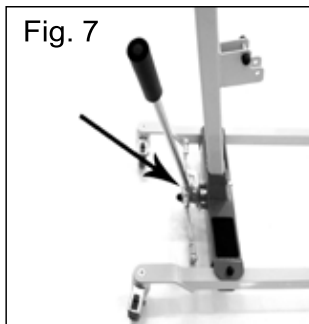
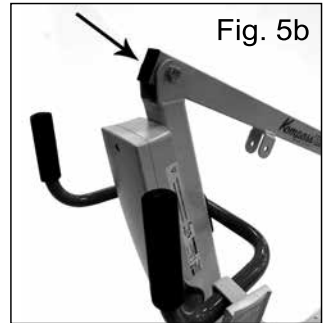
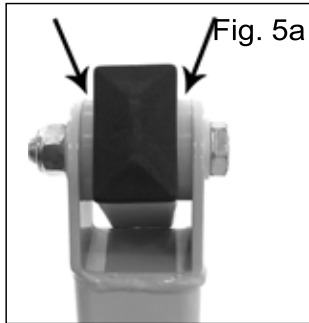
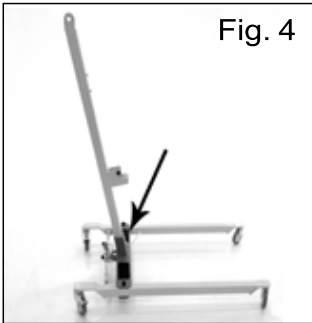
10. ASSEMBLY INSTRUCTIONS FOR LIFT STRUCTURES (for all the models)

1. Unscrew the screw at the base of the mast.
2. Fit the mast into the provided cavity of the base. (Fig.4)
3. Fit the screw that blocks the mast into the base and screw it well. (Fig.4)
4. Connect the boom to the mast with the provided screws. (Fig.5a-5b)
5. Fit the handle into the mast and block it with the 2 screws. (Fig.6)
6. Fit the lever for opening the base and block it with its screw. (Fig.7)
7. Fit the lifting beam into the corresponding hook, placed at the end of the boom. Fit the plastic washers on both sides of the anchor bolt. (Fig.8)
8. The structure will be completely assembled. Install the hydraulic/electrical actuator suitable for the chosen patient lift

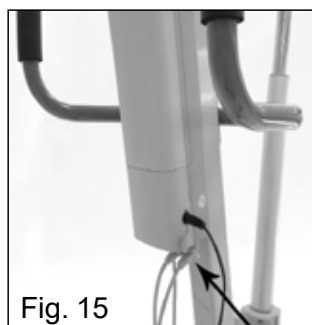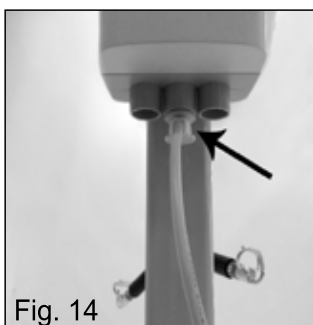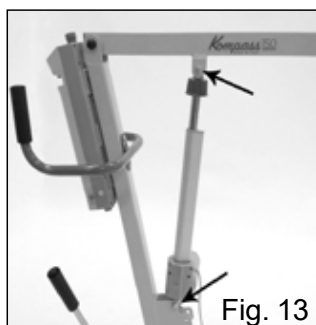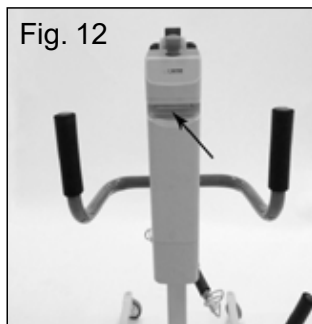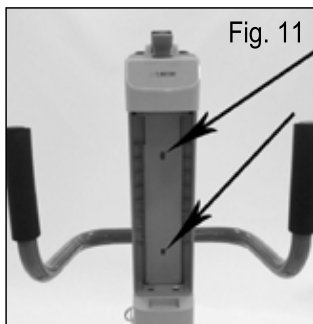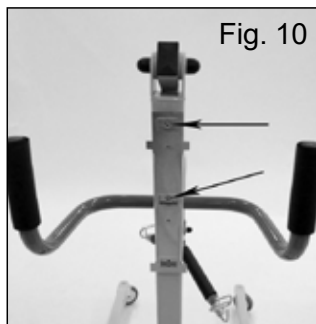11. ASSEMBLY INSTRUCTIONS TiMOTION ACTUATORS (for all the models)

1. Secure the support mount of the control unit to the mast with the provided screws, using the fitted cavities. (Fig.10)
2. As soon as you have assembled the mount, fit the control unit on it, letting it slide from top to bottom and blocking it with the screws. (Fig.11)
3. Fit the battery into the control unit. (Fig.12)
4. Remove the two actuator mounting bolts from the mast boom. Fit the actuator into it and secure it with the two bolts previously removed. Cover the bolt heads with the provided plastic cap. (Fig.13)
5. Wire up the actuator connector to the central socket of the control unit and the remote control to the right socket. (Fig.14-15)

12. INSTRUCTIONS FOR USING TiMOTION ACTUATOR LIFTS (for all the models)

BEFORE USING THE ELECTRICAL LIFT YOU ARE RECOMMENDED TO CHARGE THE BATTERY FOR ABOUT 6 HOURS

1. Before lifting a patient, open the base legs to the maximum width possible, gently pulling the opening lever towards you and secure it in the open position (only if there is a mechanical opening of the base legs)
2. Block the rear wheel brakes
3. Keep the patient centred between the base legs and always turned towards the assistant
4. Hook the cradle straps securely to the lifting beam

5. Press the up-facing arrow key in order to lift the patient (for the model Talos the led display on the control unit will show the boom moving)
6. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!
7. Lift the patient to the desired height, keeping him facing his assistant at all times
8. Release the rear wheels and move the patient
9. Before lowering the patient, block the rear wheels again
10. In order to lower the patient, press the down-facing arrow key (for the model Talos the led display on the control unit will show the boom moving)
11. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!

ALWAYS HELP THE PATIENT DURING THE LOWERING PROCESS SO AS TO MAINTAIN THE CORRECT SEATED POSITION!

WARNING!

While lowering the patient, please remove any barrier that can impede the downward movement. If the case, lift up the patient again and remove what causes the impediment for lowering.

AUTOMATIC ANTI-CRUSHING SYSTEM

The electrical kit is equipped with an automatic anti-crushing system, which stops the lowering of the lifting boom if there is an accidental contact between the patient and any barrier. In this case, you can hear a "bip" and the lowering process will be stopped immediately (in the model Talos the led display will show the word "OBSTACLE"). In order to let the process start again, remove the obstacle and press again the down-facing arrow key on the remote control.

EMERGENCY STOP

The control unit is equipped with a red "STOP" button which instantly deactivates all controls when pressed. To restore normal operation, turn the button clockwise and pull it outwards.

EMERGENCY KEYS

On the control unit there are two keys (one with an up-facing arrow and the other one with a down-facing arrow) that can be pressed instead of those ones on the remote control to complete all the operations when the remote control fails to work properly.

COLOR DISPLAY (Only for the model Talos)

This model is equipped with an electronic control unit with color graphic display.

LEFT KEY: Pressing the left key for more than 0,5 seconds, the boom lifting function can be activated		RIGHT KEY: Pressing the right key for more than 0,5 seconds, the boom lowering function can be activated
---	---	--

If one of the two buttons is pressed for more than 0,5 seconds, one of the following functions will be activated:

		
Activation lifting boom (valid for all the models)	Electrical activation base legs opening (only with an electrical actuator for opening base legs)	Residual battery capacity

MAINTENANCE	MAINTENANCE
	
Months	Cycles
Time of use in MONTHS	Cycles done

NOTICE (Only for the model 0804902 TALOS)

ATTENTION	ATTENTION	ATTENTION
 Recharge Or Exchange	 Overloaded	 Please Check Hoist
Low battery capacity Recharge or replace the battery	Excess load alarm The load lifted by the patient lift exceeds the maximum level allowed. Remove the load and try again.	Anti-crushing safety system In case of accidental contact between the patient and any barrier, the boom lowering process will be automatically stopped. Remove the obstacle and keep on.

MECHANICAL LOWERING OF THE ACTUATOR

If using the manual/mechanical lowering, please make sure that it will be done in total safety conditions for the patient. In order to activate the manual/mechanical lowering, pull downwards the red ring placed in the upper part of the actuator and turn it counter-clockwise. Once finished the lowering process, take the red ring back to its resting position.

INDICATION OF THE BATTERY STATE OF CHARGE

There is a 3-Led indicator for the battery state of charge. This indicator is placed on the remote control. 3 Leds on indicate a complete charge of the battery, whereas one just Led on indicates the battery is out of power and needs to be charged.

RECHARGING BATTERIES

- Connect the adapter to the main socket and then to the socket on the control unit, after removing the protection cap
- On the charger adapter there is a led that during the charging process becomes orange color. Charge the battery until the led becomes green color.
- Before activating the lift again, make sure to disconnect the battery charger!
- Charge the batteries for at least 6 hours before using the lift

13. WARNINGS

- The lift is a device designed for moving patients with motor problems for short distances, for instance from a bed to a wheelchair or comfortable chair, and back. It must not be used for transporting the patient.
- While lifting the patient, make sure the base legs are as far apart as possible, the control lever is securely locked and the rear wheel brakes are secured.
- While moving the patient, please avoid crossing barriers that might cause the lift to tip over.

WARNING!

Always check that the cradle is in good condition. Do not use cradles showing signs of damage or wear.

14. CLEANING

To clean the cradle and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

15. MAINTENANCE

- It is extremely important to check the lift before each use.
- More specifically, please check that all screws and bolts are well tightened.
- Please, check the lift can move freely with the wheels unlocked by the brakes.
- Check the rear wheel brakes function correctly.
- It is very important to regularly check the connection pins and stop rings are correctly located (the pins connect the actuator to the upper and lower part of the lift, and the lifting beam where you hook the cradle).
- The mechanical parts must be lubricated with a drop of oil in order to facilitate their sliding and resistance to wear. In particular, pay attention to the mechanical retractor for the legs of the patient lift. Lubricate as indicated above every two or three months.

16. SPARE PARTS

For spare parts and accessories, please always refer to the MOBIAK S.A master catalogue.

ONLY USE ORIGINAL MOBIAK S.A ACCESSORIES. MOBIAK S.A COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

17. DISPOSAL OF BATTERIES AND ELECTRONIC COMPONENTS

a. Warnings for the proper disposal of the product, as required by the European Directive 2002/96/EC

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

b. Treatment of dead batteries (Directive 2006/66/CE)

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from whom you purchased the device.

18. WARRANTY

Every MOBIAK S.A product is guaranteed for 3 (three) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. MOBIAK S.A shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. MOBIAK S.A does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by MOBIAK S.A, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from MOBIAK S.A original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

a. Warranty for rechargeable batteries (if provided)

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

b. Reparations

If a MOBIAK S.A product shows material or manufacturing faults during the warranty period, MOBIAK S.A, together with the client, will consider whether the product fault is covered by warranty. MOBIAK S.A, at its unquestionable discretion, may replace or repair the item under warranty in a MOBIAK S.A reseller point or at its own qualified offices. Labour costs for repairing the product may be at MOBIAK S.A's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from MOBIAK S.A client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

The client is informed if, after examining and testing a returning product, MOBIAK S.A decides that the product is not defective, the product will be sent back to the client and all the return delivery costs will be at his expense.

c. Spare parts

The original MOBIAK S.A spare parts are guaranteed for 6 (six) months from the day you receive them.

d. Exoneration clauses

Except what clearly specified in this warranty and in accordance with the law, MOBIAK S.A does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. MOBIAK S.A does not guarantee that the use of MOBIAK S.A products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

Notes

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

WARRANTY CERTIFICATE

Product _____ S/N _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

Kathiana-Akrotiriou, 73100, Chania, Crete, Greece - Tel. +30 28210 63222,
www.mobiakcare.com email: mobiakcaresales@mobiak.com