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PRODUCT DESCRIPTION

The air mattresses can reduce and treat decubitus ulcers in chronic patients or people with long stay in bed. They can be used for hospital or home use. Evenly distributes the pressure of the body to avoid the generation of edema. The pump switches the air pressure in the mattress, creating a feeling of light massage and activates the immobilized cells due to the sprawling

For the SET **0810160** features micro-holes opened with LASER technology, not visible at various points, allowing air to reach the user and preventing moisture sores

Contents: Air-Pump, Mattress, Tubes, Glue, Patch, Filter, Fuse.

OPERATION USE

For the mattress (0808279 or 0810561):

- ☐ Place the air mattress over the bed mattress, having first stretch bottom sheet, making sure the air inlet is at the side of the legs.
- ☐ Place over from one minute air mattress breathable cloth, if not accompanied by its own cover, so as not to come into direct contact with the patient's skin.
- ☐ Place the extra parts under the bed mattress.

For the pump (0808578)

- ☐ Attach the mattress to the pump using appropriate tubes are in the package. Connect the pump to the mains and turn the knob to the maximum in order to inflate the mattress.
- ☐ When the mattress inflates completely you can adjust the air flow, depending on the user's requirements, using the switch.

SPECIFICATIONS

PUMP	Electric	AC220-240V 50/60HZ
	Output	4-6 Lit / minute
	Pressure Range	30mmHg-120mmHg
	Exchange Cycle	10-12 minutes
	Power	≤ 10VA +25%
	Noise	≤ 50 dB
	Pump Dimensions (LxWxH):	28x15,5x16 cm
STRIPE MATTRESS	Arrangement	Air Stripes
	Material	Medical Grade PVC
	Dimensions	200x95x11 cm
	Max Load	150Kg
	Product Weight	7 Kg

SPARE PARTS

REF. NUMBER	DESCRIPTION
0808578	PUMP FOR AIR-STRIPE MATTRESS
0808279	SPARE STRIPE MATTRESS
0810561	SPARE STRIPE MATTRESS WITH LASER-HOLES
0810078	SPARE FILTER FOR AIR-PUMP

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$ deviation for the measurements on the Products.

MAINTENANCE

- ☐ Do not disconnect the hoses while the pump is running.
- ☐ Do not use sharp objects near the mattress.
- ☐ Do not use any flame hearth near the mattress.
- ☐ Do not exceed the maximum load (kg) recommended by the manufacturer.
- ☐ Keep the pump in a clean & dry environment.
- ☐ To clean the mattress, use a soft cloth or sponge and soapy water. Do not use corrosive, acid or other chemical cleaners.
- ☐ The pump has security RF1 05x20, 1,5A. If you need to replace it, disconnect the pump from the mains and using a screwdriver turn the cap of the fuse in contrary to the clockwise direction (left).
- ☐ The pump has a dust filter. Please check at least once a week that the filter is clean. If necessary, wash the filter with water, dry it thoroughly and reposition it in its place. If the device operates with dirty filter can lead to malfunction or damage the device. The filter should be checked regularly.
- ☐ If you need to save the mattress, empty of air, folding it.
- ☐ It should be stored in dry, clean and well-ventilated environment.

IMPORTANT (!)

CAUTIONS

- ❑ Protect the product by oils, lotions and creams, by covering it with the cotton wool fabric, or a bath towel.
- ❑ Do not use abrasive alcohol-based cleaning when the mattress. This could damage the material of the layer.
- ❑ Clean the upholstery after each daily use. We recommend that you clean the upholstery with a mild detergent to protect it from oil stains and dust. A common practice is to use a spray bottle filled with a mixture of anti-bacterial soap and water. Spray the surface and then wipe with a clean towel. The best solution is to be cleaned with warm water mixed with a mild soap and then thoroughly dried.
- ❑ Protect the layer from scratches, cuts and punctures.

GENERAL WARNINGS:

- ❑ **This manual MUST be given to the user of the product.**
- ❑ If you don't read this user manual, it is preferable not to use this product or another available part.
If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- ❑ READ this user manual and SAVE for future reference.
- ❑ Keep away the product from heat sources.
- ❑ Be careful when children are nearby and do not allow the children to play with the product.
- ❑ Do not exceed the maximum payload.
- ❑ Special care is always required where there are moving parts that can cause limb entrapment and injury.
- ❑ Do not attempt to lift the product by any removable parts.

Environmental Protection

If you find one day that your appliance needs replacement, or no longer serves you, consider the Environment:

- 1). Do not dispose of your device along with the municipal waste (this is also the importance of the stated recycling symbol).
- 2). Contact your municipal authority to indicate the point of disposal of your device for recycling.
- 3). With the most useless your device in the right place
Recycling helps to protect the environment and the operation again your device materials.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY

The product has warranty for two (2) years in pump & six (6) months in the mattress, from the purchase date.

The warranty covers factory defects & does NOT cover defects resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product instructions for use. It also does NOT cover worn parts and whatever it is going to be damaged during use or over time.

Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and transportation companies are not covered by the warranty.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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