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T: +30 28210 63222 | WEB: www.mobiakcare.com

## PRODUCT DESCRIPTION-INTENDED USE

This aluminum stick can be used by people for walking assistance.

The foldable stick can be assembled easily joining the 4 pieces which are included.

For the adjustable stick, untighten the safety washer to adjust the height and tighten again for the stabilization.

For the Stick with the light (code 0810207) unscrew the socket behind the handle & put one battery AA so the light can work.

It has plastic storage packing

Frame construction material: Aluminum

## SAFETY ADVICE

- Regularly clean the stick
- Do not let the stick wet.
- Check periodically the stick and if you find a damage please fix it at an authorized partner
- Do not use unauthorized parts other than those approved by Mobiak
- Don't put weight on the stick over than the permissible.

## CLEANING

- Use cold water and mild soap to clean the stick
- Dry it with a soft cloth
- Dirt, oil or debris must be removed from the stick

## MAINTENANCE

- Clean the stick after each use with a soft cloth.
- Do a complete test - usage, before its use
- Check the ferrule rubber tip often and change it if it is worn. Spare part code for the ferrule rubber tip is 0809363 and 0810599 for the stick with the light.

## DECLARATION OF CONFORMITY

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

## ATTENTION (!)

- In case of failure or malfunction of your product, please contact your authorized and trained dealer.
- Use of the following product is prohibited for purposes other than those specified in this manual
- MOBIAK SA disclaims any responsibility for damage resulting from improper use of the product or use other than that set forth in this manual.
- MOBIAK SA reserves the right to make changes to the product and the following manual without notice in order to improve its features.

## ENVIROMENTAL PROTECTION

If your product needs replacement ,or you do not use it, consider the environment protection:

- 1) Do not dispose your product with other waste (this the meaning of the recycled symbol).
- 2) Contact your local authority to instruct you the recycling centers for your product.
- 3) Recycling your no use product, you protect the environment and help the exploitation of new materials from your product .

## **WARRANTY**

The product has warranty for one (1) year from the purchase date.

The warranty covers factory defects and not damages resulting from improper use / maintenance of the product.

The warranty apply factory defects & DOES NOT apply damages resulting from improper use , poor maintenance, modification, abuse or non-compliance with the product's operating instructions.

Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and transportation companies are not covered by the warranty.

### **Warranty Form**

#### **Buyers Data**

Name: .....

Date of purchase: ..... S/N .....

LOT:..... Phone: .....

#### **Resellers Data**

Name :

.....  
.....

Address:.....

.....

Phone: .....

**Reseller Stamp**

### **INCIDENT REPORTING:**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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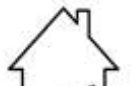
Description of symbols that have been printed on the label, Master Carton and or User Manual

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MOBIAK S.A