GR EN

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INTENDED USE

The "ASPER" wheelchair is used firstly as a toilet and secondly as movement aid for people with limited mobility for indoor use.

It can be used in 2 ways:

- as a toilet seat
- as a raised toilet seat (without the container, for use above the toilet of the house)

SPECIFICATIONS

FRAME	Aluminum
COLOR	Black
SEAT	PU
SIDEBOARD	Fixed
FOOT SUPPORT	Steady / Foldable
REAR CASTORS	4" Compact
FRONT CASTORS	4" Compact
BRAKES	Only on rear castors

DIMENSIONS

0805454	44cm	43cm	43cm	45cm	53cm	65cm	97cm	53cm	4"4"	136kg	9kg

SYMBOLS

Please refer on **ANNEX A**

ASSEMBLY

Please refer on **ANNEX B**

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

MAINTENANCE

The purpose of periodic check of the toilet wheel, is to confirm its proper operation as an aid and must be done at least once a month.

- Check the container status for any breaks, cracks, or leaks.
- Check the steel frame screws. In case you find loose please tighten it.
- Check plastic parts for looseness, cracks or breaks.
- Check stability (without wheels) or balance (with wheels) of the toilet.
- Check the wheel brakes and confirm their function.
- Check the condition of the front and rear wheels.

In case of repair, use only authentic spare parts.

CLEANING

Clean the toilet container immediately after use. Initially, rinse with warm water. Then use a soft cloth impregnated in a mild detergent or antiseptic solution. Wipe with a clean, soft cloth.

CAUTION: Do not use acidic or caustic solutions.

We recommend that you clean the steel frame and the seat with a mild detergent or antiseptic solution. For your own convenience, you can use a spray bottle after adding the solution. Spray the product and then clean with sponge. With a clean cloth, pass the surfaces to dry.

IMPORTANT(!)

Before using the commode chair, make sure:

- It is in full extent & secured in this position.
- Wheels (if available) are well secured.
- The seat is properly applied & secured in this position.
- The container is correctly positioned in the appropriate slot.
- There are no cracks or breaks on the skeleton.
- Do not smoke & do not use flame near the product.
- Protect the product from abrasions, cuts and jabs.

OTHER INSTRUCTIONS

- When sitting or raised, you must press the handles down so that the toilet stays still and not slipping.
- When you are going to get up you need to rely on both handles. Support only in one hand (and without companion) can cause accident with damage or injury.
- Regularly clean the toilet.
- Do not leave the toilet wet.
- Do not let children near the toilet without supervision.

WARNING

In case of problem, do not use the wheelchair until the problem is solved.

- Do not use the product for a purpose not indicated in this manual.
- The manufacturer declines all responsibilities for any consequences resulting from an incorrect use of this product.
- The manufacturer reserves the right to change the information contained in this document without notice.
- Do not allow children to play on the wheelchair.

- Do not attempt to lift the wheelchair by any removable parts.
- Keep the packed wheelchair away from heat sources.
- Getting in and out of the wheelchair may be performed independently, provided you have an adequate mobility and upper body strength.

ENVIRONMENTAL PROTECTION

Should you find one day that your product needs to be replaced or if it is of no further use to you, think of the protection of the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your appliances must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product components.

WARRANTY

Your wheelchair is guaranteed for 2 years from the purchase date.

The warranty covers factory defects & does NOT cover defects resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product instructions for use. It also does NOT cover worn parts during use. Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where provided), transport companies are also not covered by the warranty. Shipping costs for goods of warranty are to be paid by the customer or by dealer.

WARRANTY CLAIM FORM

BUYERS DETAILS			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SELLER DETAILS			
FULL NAME		ADDRESS	
PHONE		SIGN AND SIGNATURE	

E.C. Declaration of Conformity

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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Required Toolset for Assembly:
Key No 10.
Allen Key No 4.

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Adjust the backrest position so the holes are
inline.

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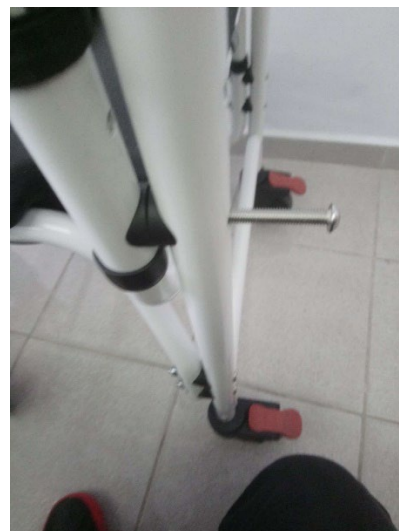
Place the spacers like the photo. Then insert
them between the pipes.

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Insert the screw throughout the aligned
holes.

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Repeat the procedure for the rest 3 nuts.