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## WARNINGS AND PRECAUTIONS

- Please keep this user's manual for future reference.
- Always touch the tube at the cuffs, when connecting or disconnecting the therapy device, humidifier and mask.
- The tube must not be heated (>60° C / >140° F) or sterilized.
- The tube may be used by one patient only.
- After washing, the tube must not be dried using a (microwave) oven, a hairdryer or other appliances. See cleaning instructions.
- The interior of the tube can only be cleaned with soft brushes or other objects, as this can damage the tube.
- Do not disassemble the tube or damage it.
- Do not insert or drop any object in the openings of the tube as it can damage the tube.
- Do not place objects on the tube as it can damage the tube.
- If the tube is damaged (holes, creases, tears, etc.) or no longer functional, do not try to repair it. Replace the tube immediately.
- The tube should not be used for longer than 12 months and should be replaced once this period is over.
- Label the old tube so that it is not used by another patient before its final disposal.

## INTENDED USE

The tube is intended for use with respiratory therapy devices and humidifiers with conical ISO connectors (ISO 5356-1). The tube is intended to transport breathing gases from the flow generator or humidifier to the patient. The tube is indicated for non-invasive respiratory therapy in the home, hospital or sleep-lab setting by a single adult patient. It can also be used in conjunction with supplemental Oxygen. The hose may only be used by adult patients

**Operating conditions:**  
Ambient temperature: 5 °C - 40 °C (41 °F - 104 °F) Relative humidity: 5% - 95% Maximum operating pressure: 60 hPa

## In case of assistance, unexpected operation or events

In case an unexpected operation or an unexpected event or for additional assistance or information, with regards to this product, get in contact with the vendor

## CLEANING

### Warnings:

1. Do not use harsh soaps, detergents, solvents or alcohols to clean the tube.
2. Do not clean the tube in a dishwasher.

### Cleaning Instructions:

1. Clean the tube in lukewarm, mildly soapy water.
2. Rinse the tube with clean water.
3. Hang-up the tube with both cuffs hanging downwards.
4. Allow the tube to dry.

### Product specification - Storage and transport conditions:

See product label

## DISPOSAL

 The tube, packaging and the required user's manual can be disposed of with the household waste.

## WARRANTY

The tube is a consumable and must be checked before first use otherwise warranty claims are excluded. Warranty expires, if:

- Interventions, modifications, extensions, calibration, repairs and maintenance are carried out by persons who are not authorized by the manufacturer,
- Other manufacturers' accessories and spare parts are used that have not been approved by the manufacturer to be used on the product
- The product is used other than as described in the user's manual
- Wearing parts are worn,
- Product damages are caused by force majeure (e.g. by lightning),
- The hygiene and cleaning instructions described in the user's manual have not been complied with.

Statutory warranty rights remain unaffected by this.

### Important notice to the user and / or patient:

Users and/or patients should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

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