MOPEDIA®

MA B1100_B1110_B1180_B1200 05 A_ITA_06 2021

BARELLA DOCCIA NEFTI

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ITALIANO

INDICE

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ENGLISH

INDEX

1. CODES	PAG.3
2. INTRODUCTION	PAG.3
3. INTENDED USE	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
4.1 Standards and reference directives	pag. 4
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION	PAG.5
8. BEFORE EACH USE	PAG.6
9. USE MODE	PAG.6
9.1 Use instructions models hydraulic BI100-BI180 (hydraulic/pneumatic)	pag. 6
9.2 Use mode model ELECTRIC TIMOTION BI110 (electric)	pag. 7
9.3 Use mode model ELECTRIC TIMOTION BI200 (electric)	pag. 7
9.4 How recharge the batteries BI110	pag. 7
9.5 How recharge the batteries BI200	pag. 8
9.6 Use of folding sidewalls BI100-BI110-BI180-BI200.....	pag. 8
9.7 Use of wheels with directional/total lock BI100-BI110-BI180-BI200	pag. 9
10. MAINTENANCE	PAG.9
11. CLEANING AND DISINFECTION	PAG.9
11.1 Cleaning.....	pag. 9
11.2 Disinfection	pag. 10
12. DISPOSAL TERMS	PAG.10
12.1 Warnings for the proper disposal of the product, as required.....	pag. 10
by the European Directive 2012/19/UE	pag. 10
12.2 Treatment of dead batteries.....	pag. 10
(Directive 2006/66/CE).....	pag. 10
13. DECLARATION OF ELECTROMAGNETIC COMPATIBILITY	PAG.10
14. SPARE PARTS	PAG.13
15. TECHNICAL SPECIFICATIONS	PAG.13
15.1 General specifications	pag. 13
15.2 General specifications for hydraulic trolley Nefti bi100.....	pag. 14
15.3 General specifications for hydraulic trolley Nefti BI110	pag. 14
15.4 General specifications for pediatric hydraulic trolley Nefti Small BI180.....	pag. 14
15.5 General specifications for bariatric electric trolley Nefti Big BI200	pag. 15
16. WARRANTY	PAG.15
16.1 Warranty of recyclable batteries bi110-bi200	pag. 15
17. REPAIRS	PAG.15
18. SPARE PARTS	PAG.16
19. NON-LIABILITY CLAUSE	PAG.16

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

- BI100** Shower trolley **NEFTI HYDRAULIC**, max weight (water included) **180 kg**
BI110 Shower trolley **NEFTI ELECTRIC**, max weight (water included) **200 kg**
BI180 Shower trolley pediatric **NEFTI SMALL HYDRAULIC**, max weight (water included) **180 kg**
BI200 Shower trolley pediatric **NEFTI BIG ELECTRIC**, max weight (water included) **325 kg**

2. INTRODUCTION

Their design and quality ensure the maximum comfort, safety and reliability. **MOPEDIA shower trolleys Nefti by Moretti S.p.A** are designed and manufactured to meet all your needs for a practical, correct and safe use. This manual contains small suggestions for proper use of the device you have chosen and valuable advice for your security. You may read the complete manual before using the shower trolley. In case of doubts please contact your dealer, who will help and advice you properly.

3. INTENDED USE

MOPEDIA shower trolleys Nefti by Moretti S.p.A are intended for patients care during washing/showering when horizontally oriented or inclined (up to a maximum of 5°, 7° only BI200), with the help of an assistant that operates commands and executes the washing operations. It cannot be used by a single person.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of SHOWER TROLLEYS complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

4.1 Standards and reference directives

To ensure the security standards for users, Moretti S.p.A. tests and respects the directive:

- UNI EN 12182 (PRODUCTS INTENDED FOR PEOPLE WITH DISABILITIES ASSISTANCE)

5. GENERAL WARNINGS

WARNING!

- Always pay close attention of moving parts that could cause arms entrapment limbs and injuries
- Always check the presence of damage to the structure that could compromise the proper safe operation of the product.
- For the correct use of the device, please refer to the present manual, giving particular attention to the Maintenance and warnings before each use.
- For the correct use of the device, consult your doctor or therapist.
- Maintain the product packed away from heat sources because the packaging is made of carton
- The service life of the device is determined by wear of parts not repairable and/or replaceable.
- Always pay attention to the presence of children.
- Do not exceed the weight limit, washing water included.
- Do not open the sides, neither during the transport nor during the patient washing.
- Do not clean with aggressive products such as ammonia, bleach, acids and abrasive detergents.
- Do not load the patient weight on the sides of the bathing plane.
- MOPEDIA shower trolleys by Nefti must be used only by properly informed medical staff.
- Set the trolley in horizontal position before let a patient sit on/go down.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Attention, should read the instructions

Production dates

Waste Electrical & Electronic Equipment (WEEE)

IP54 Protection against dust and water splashes

B Type

7. GENERAL DESCRIPTION

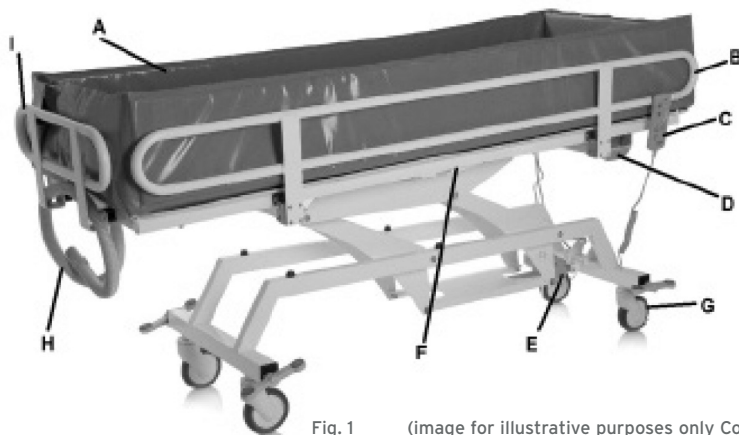
Fig. 1 (image for illustrative purposes only Cod. BI100-BI110-BI180)

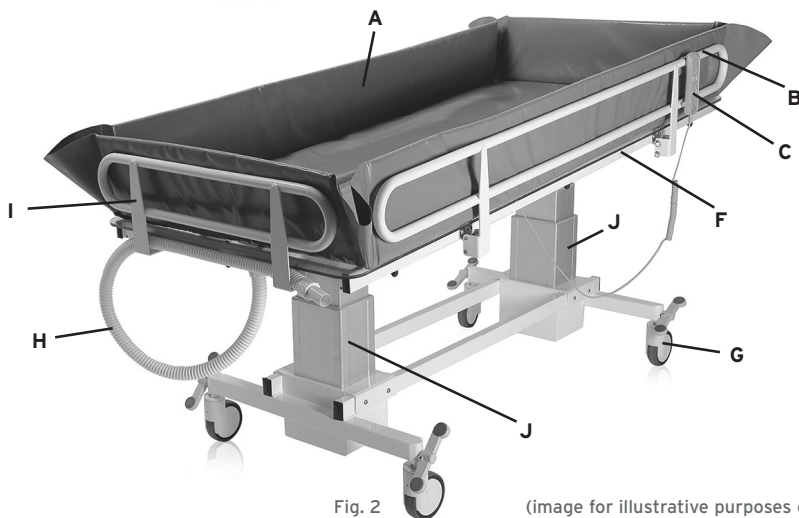
Fig. 2 (image for illustrative purposes only Cod. BI200)

- A. Washing towel
- B. 180° foldable side rail
- C. Motor unit remote control (only for BI110-BI200)
- D. Control unit with battery (only for BI110-BI200)
- E. Lifting motor unit of bathing plan (hydraulic BI100-BI180, electric BI110)

- F. Tilt motor unit of bathing plan(gas spring BI100-BI180, electric BI110)
- G. Wheels with directional/total lock
- H. Drainage pipe
- I. Head/foot side locked rail
- A. Column actuators for elevation and inclination of the bathing plan (BI200)

8. BEFORE EACH USE

- Always check the correct closing of the lateral containment sides. In case it is not possible to close the sides the product must not be used.
- Before patient washing or transferring, verify that the wheels are braked and the sides are lifted;
- During the transfer, avoid passing through obstacles that could cause an overturning.


WARNING!


- Verify the good condition of the washing towel and the drainage pipe, the damages caused by the wear can produce water leaks during the washing

9. USE MODE


9.1 Use instructions models hydraulic BI100-BI180 (hydraulic/pneumatic)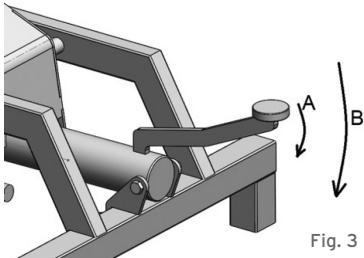

- To raise the plan of the trolley lifting, press many times the raiser pedal like in the fig X (rif A), to lower the bed press totally the raiser pedal (rif B)
- To calibrate the grade of the washing plan, unblock the gas spring using the lever at the center of washing plan and execute the grade (do this operation without the load on the trolley)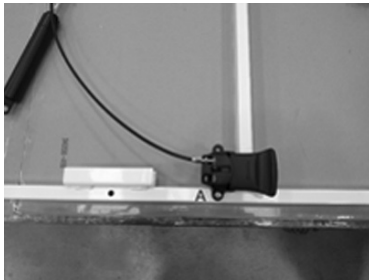

- Unscrew the bolt in picture (A), turn the handle in clockwise and remove the screw (hidden by the handle in picture) from the respective hole.

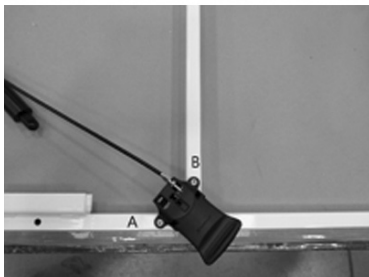- Place the handle in order to match the holes (of the handle and plan too) and subsequently tighten the screws (A and B) mildly (to avoid to damage the handle structure).

9.2 Use mode model ELECTRIC TIMOTION BI110 (electric)

- Before each use verify that the battery is correctly inserted in the control unit special seat and that is charged, cannot be charged the batteries during the use.
- Press the button 1 to increase the washing plan height
- Press the button 2 to decrease the washing plan height
- Press the button 3 to increase the washing plan grade
- Press the button 4 to decrease the washing plan grade
- Emergency stop: on the control unit there is a red button "STOP", if you press it you can block all the orders. To reactive the functioning turn the red button clockwise and make it redeem out.

Fig. 4

9.3 Use mode model ELECTRIC TIMOTION BIZ00 (electric)

- Before each use verify that the battery is correctly inserted in the control unit special seat and charged, batteries cannot be charged during the use. Press the "ON" button on the control unit for about 2 (two) seconds to turn on the shower trolley, press the "OFF" button for about 4 (four) seconds to turn off the shower trolley.

- Press the button 1 to increase the washing plan height
- Press the button 2 to decrease the washing plan height
- Press the button 3 to increase the washing plan grade
- Press the button 4 to decrease the washing plan grade

9.4 How recharge the batteries BI110

NOTES: ensure that the red emergency button is turned off .To do this turn and pull to yourself the RED button.

- Connect the adapter to the power outlet and, so, to the input of the control unit, after you have removed the relative protection cap.
- On the charger adapter there is a led that during the charging phase is orange. Charge the battery until the led becomes green.
- Before to use the shower trolley remember to detach the adapter from the control unit.
- Charge the batteries for at least 6 hours before using the shower trolley.

9.5 How recharge the batteries BI200

NOTES: ensure that the red emergency button is turned off. To do this turn and pull to yourself the RED button.

- Connect the adapter to the power outlet and, so, to the input of the control unit (FORCEFULLY UNTIL THE END), after you have removed the relative protection cap.
- On the charger adapter there is a led that during the charging phase is orange. Charge the battery until the led becomes green.
- Before use the shower trolley, remember to detach the adapter from the control unit.
- Charge the batteries for at least 6 hours before using the shower trolley

9.6 Use of folding sidewalls BI100-BI110-BI180-BI200

The folding side rails are equipped with a 3-position support with a safety guide against accidental release, which supports the folding sides and allows them to be securely and correctly fixed in the 3 foreseen positions.

Description of sides positions and instructions for their handling:

- Side panels in raised position, with pins inserted in the "internal" notch (Fig.5 and 5a). Slightly lift the side rails using both hands to unlock.
- Side rails in an inclined position, reached by slightly lifting the side rails and pulling them slightly backwards, inserting the pins in the "external" notch (Fig. 6). Slightly lift the side rails using both hands to unlock.

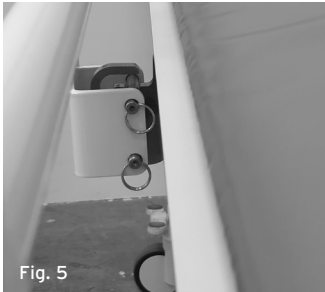
Fig. 5

Fig. 5a

Fig. 6

- Side rails in lowered position, reached by slightly lifting the side rails and making them rotate forward until, when passing the safety guide, it is possible to extract the side rails by further lifting by rotating them backwards and accompanying the rotation movement out of the support until reaching the complete lowering avoiding shocks and sudden movements (Fig.7, 8)

Fig. 7

Fig. 8

- To close the side panel, carry out the operations indicated above in reverse order.

WARNING!

- Always check the correct closing of the sides through the locking pin.
- The operator shall not lean with his weight on the sides.
- Do not use the sides as a support for the patient's movement.
- Always move the side rail with 2 hands, and always make sure that pins of the 2 sides proceed in parallel, and maintain the same position both during movement and at rest, (for left and right brackets) and that if raised or tilted both pins are positioned in the same notch.

9.7 Use of wheels with directional/total lock BI100-BI110-BI180-BI200

- Press the pedal until you hear the first "click" to obtain the directional lock (for long and straight transfers)
- Press the pedal until you hear the second "click" to obtain the total lock of wheels
- For a reliable braking, totally lock the wheels on both the axes of the trolley.

10. MAINTENANCE

When Moped by Moretti accessories are placed in commerce they are carefully checked and provided of CE brand. For patient and doctor safety we recommend that the eligibility of your device must be checked, at least every 2 years ,by the producer or by an authorized laboratory. In case of repair must be used only original spare part and original accessories.

- Is very important that the shower trolley is checked before every use. In particular we please to check that all screws and bolts are well tightened.
- Is important that the shower trolley can move freely with the wheels brakes deactivated
- For a correct use verify the wheels brakes.
- Is very important to verify periodically that the connecting pins/mechanical joints and relative blocking rings are well placed.
- The mechanic parts must be lubricated with oil to help the fluidity and wearing resistance. Keep close attention to the lifting/tilting mechanic device of the washing plan. Lubricate as indicated above every 2 or 3 months.

11. CLEANING AND DISINFECTION

11.1 Cleaning

Use a damp cloth and neutral detergent to clean the device's plastic and metal parts. Do not use products which can scratch the trolley.

WARNING!

Do not use abrasive substances, acids, alcohol, chlorine-based cleaners, disinfectants and acetone to clean the trolley, because these can cause abrasions of plastic parts and rusting of metallic areas.

The manufacturer won't be responsible for damages caused by usage of materials that can corrode the areas of the product or chemical corrosive during cleaning.

11.2 Disinfection

If needed, perform the sanitization with a mild disinfectant detergent

12. DISPOSAL TERMS

In case of device disposal never use an urban rubbish container. Is recommended to dispose the device through the common municipal ecological islands.

12.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

12.2 Treatment of dead batteries (Directive 2006/66/CE)

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from whom you purchased the device.

13. DECLARATION OF ELECTROMAGNETIC COMPATIBILITY

The NEFTI shower trolley device is designed for use in the electromagnetic conditions specified in the following tables. The NEFTI device user must be sure that these conditions, during the usage, are real.

Table 1

Manufacturer's guide and declaration - electromagnetic emissions		
The NEFTI device is intended to be used in the following electromagnetic conditions. The client or user must ensure that it is used in these conditions.		
Emission tests	Conformity	Electromagnetic environment - guide
RF emissions CISPR 11	Group 1	The NEFTI device uses RF energy only for charging the battery. Its RF emissions are therefore very low and in all likelihood do not interfere with nearby electronic equipment.
RF emissions CISPR 11	Class B	The NEFTI device is intended to be used in all buildings, including the home and buildings connected directly to the low voltage public power grid for residential applications.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuation/flicker emissions IEC 61000-3-3	Conforming	

Table 2

Manufacturer's guide and declaration - electromagnetic emissions			
The NEFTI device is intended to be used in the following electromagnetic conditions. The client or user must ensure that it is used in these conditions.			
Immunity tests	Test level IEC 60601	Conformity level	Electromagnetic environment - guide
Electrostatic discharge (ESD) IEC 61000-4-2	+ 6 kV contact + 8 kV air	+ 6 kV contact + 8 kV air	Information for using the device and charging the battery. The floor must be wood, concrete or ceramic. If the floor is covered with a synthetic material, the relative humidity must be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	+2KV for power lines +1 kV for I/O lines	2KV for power lines +1 kV for I/O lines	The quality of the mains supply must be that of a typical commercial hospital when charging the battery
Surge IEC 61000-4-5	1 kV differential mode +2 KV common mode	1 kV differential mode +2 KV common mode	The quality of the mains supply must be that of a typical commercial or hospital when charging the battery.
Voltage drop outs, brief interruptions and variations of voltage on the power supply lines. IEC 61000-4-11	<5% UT (>95% UT drop out) for 0.5 cycles 40% UT (60% UT drop out) for 5 cycles 70% UT (30% UT drop out) for 25 cycles <5% UT (>95% UT drop out) for 5 s.	<5% UT (>95% UT drop out) for 0.5 cycles 40% UT (60% UT drop out) for 5 cycles 70% UT (30% UT drop out) for 25 cycles <5% UT (>95% UT drop out) for 5 s.	The quality of the mains supply must be that of a typical commercial or hospital when charging the battery.
Mains frequency magnetic field (50/60 Hz) IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. (Applies both to using the device and charging the battery).
Note: UT is the AC mains voltage prior to application of the test level.			

Table 3

Manufacturer's guide and declaration - electromagnetic immunity			
The NEFTI device is intended to be used in the following electromagnetic conditions. The client or user must ensure that it is used in these conditions. Make sure it is used in these conditions.			
Immunity tests	Test level IEC 60601	Conformity level	Electromagnetic environment - guide
conducted RF IEC 61000-4-6	3 V eff. From 150 KHz to 80 MHz	3 V eff.	Portable and mobile RF communications equipment should be used no closer to any part of the [equipment or system], including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P}$ from 80 MHz to 800 Mhz $d = 2,3\sqrt{P}$ from 800 MHz to 2,5 GHz $d = 7,7$ m during recharge operation
radiated RF IEC 61000-4-3	3 V/m From 80 MHz to 2.5 GHz	3 V/m	where "P" is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and "d" is the recommended separation distance in metres (m). Fields strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1: At 80 MHz and 800 MHz, apply the separation distance for the higher frequency band. NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
<p>^a The intensity of the field for fixed transmitters, such as base stations for radio-telephones (cellulars and cordless) and terrestrial mobile radios, amateur radio equipment, AM and FM radio transmitters and TV transmitters may not be known precisely in theory. To establish an electromagnetic environment caused by fixed RF transmitters, one must run an on-site electromagnetic inspection. If the field intensity measured in the place in which the Nefti is used exceeds the RF level of conformity indicated above, the function of the Nefti must be monitored. If abnormalities are observed, supplementary measures may be necessary, such as re-orientation or repositioning.</p> <p>^b The field intensity over the frequency range from 150 kHz to 80 MHz should be less than 3 V/m.</p>			

Table 4

Recommended separation distances between portable and mobile radio communications equipment and NEFTI						
The NEFTI device is intended to operate in an electromagnetic environment in which RF disturbances are under control. The client or user of NEFTI can contribute to preventing EM interference by ensuring a minimum distance between the mobile and portable RF communications devices (transmitters) and the NEFTI battery charger during charging and the internal NEFTI device during its operation as indicated below, in relation to the maximum output power of the transmitters						
Tests	Separation distance as a function of the frequency of the transmitter m					
	During battery charging	During the use of NEFTI	During battery charging	During the use of NEFTI	During battery charging	During the use of NEFTI
Maximum rated output power of transmitter W	from 150 kHz to 80 MHz $d = 1,2\sqrt{P}$	from 150 kHz to 80 MHz $d = 1,2\sqrt{P}$	from 80 kHz to 800 MHz $d = 1,2\sqrt{P}$	from 80 kHz to 800 MHz $d = 1,2\sqrt{P}$	from 800 kHz to 2.5 GHz $d = 1,2\sqrt{P}$	from 800 kHz to 2.5 GHz $d = 1,2\sqrt{P}$
0.01	0.12	0.12	0.12	0.12	0.23	0.23
0.1	0.38	0.38	0.38	0.38	0.73	0.73
1	1.2	1.2	1.2	1.2	2.3	2.3
10	3.8	3.8	3.8	3.8	7.3	7.3
100	12	12	12	12	23	23
For transmitters whose maximum rated output power is not listed above, the recommended separation distance d , in metres (m), can be calculated using the equation applicable to the transmitter frequency, where P is the maximum rated output power of the transmitter, in Watt (W), as given by the transmitter's manufacturer. NOTE 1: At 80 MHz and 800 MHz, apply the separation distance for the higher frequency band. NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.						

14. SPARE PARTS

For spare parts and accessories, please always refer to the MORETTI S.P.A. master catalogue. TO ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.



15. TECHNICAL SPECIFICATIONS

15.1 General specifications

- Powder coated steel structure, with specific antirust treatment
- Pivoting wheels, with integrated brakes
- Elevating washing plane
- Reclining washing plane, for water drainage
- 180° reclinable bed side rails
- Waterproof PVC washing towel, with padding and siderails h25cm
- 1.5m long drainpipe
- Compliant to EN ISO 12182
- BI100 - BI180 max load: 180 kg (water included)
- BI110 max load: 200 kg (water included)
- BI200 max load: 325 kg (water included)
- Emergency STOP button (only on BI110)

15.2 General specifications for hydraulic trolley Nefti bi100

Dimensions and weight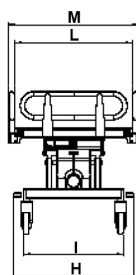
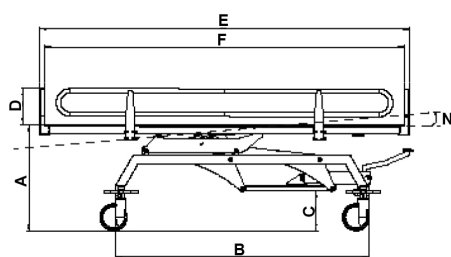


Measures in CM

A	B	C
59-93	141	23
D	E	F
20	206	200
H	I	L
67	56	66
M	N	MAX loading weight
73	0°-5°	180 Kg

15.3 General specifications for hydraulic trolley Nefti BI110

Dimensions and weight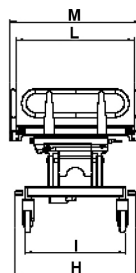
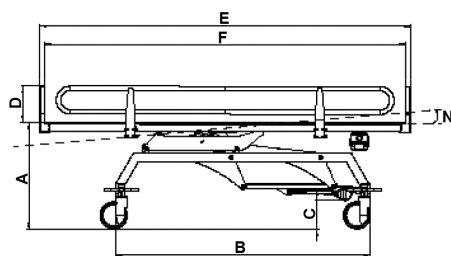


Measures in CM

A	B	C
59-99	141	16
D	E	F
20	206	200
H	I	L
67	56	66
M	N	MAX loading weight
73	0°-5°	200 Kg

15.4 General specifications for pediatric hydraulic trolley Nefti Small BI180

Dimensions and weight

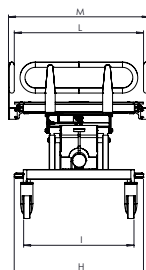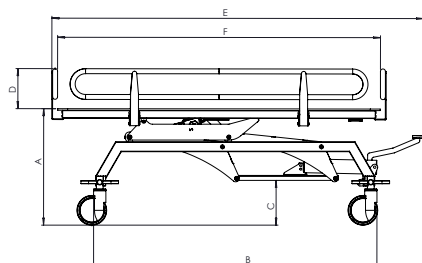
Measures in CM

A	B	C
65-91	143,5	19,5
D	E	F
20	170,5	165,5
H	I	L
65,5	56	67
M	N	MAX loading weight
72	0°-5°	180 Kg

15.5 General specifications for bariatric electric trolley Nefti Big BI200

Dimensions and weight

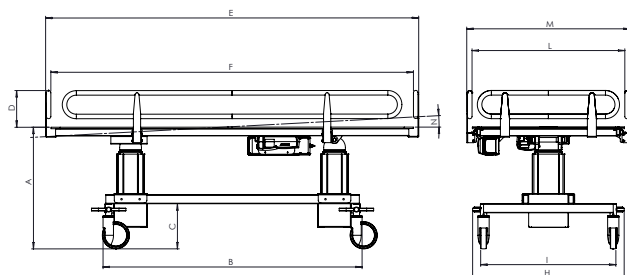Measures in CM

A	B	C
65-94	143,5	23
D	E	F
20	205	200
H	I	L
84,5	75	86
M	N	 MAX loading weight
91	0°-7°	325 Kg

16. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. Components which wear out are not covered by this warranty if the damage is due to the normal use of the product.

16.1 Warranty of recyclable batteries bi110-bi200

Original and spare batteries are covered by a warranty that lasts 90 (ninety) days in relation with their performances and a 6 (six) months warranty in relation with factory defects or law requirement. If you leave them unused completely charged more than 3 (three) consecutive months, the warranty is automatically voided. If completely uncharged batteries are left more than 3 (three) consecutive days, the warranty is automatically voided.

17. REPAIRS

Repair under warranty

If a Moretti product has material or manufacturing defects during the warranty period, Moretti will agree with the client whether the defect is covered by the warranty. Moretti, at its sole discretion, may replace or repair the article at a specified Moretti reseller or its own premises. The costs of labour incurred in repairing the product will be borne by Moretti if it determines that the repair is covered by the warranty. Repair and replacement do not renew the warranty period.

Repairing a product not covered by warranty

A product not covered by warranty may be returned for repair only if authorised in advance by Moretti customer service. The costs of labour and shipping incurred by repairs not covered by the warranty are borne by the client or reseller in their entirety. Repairs on products

not covered by the warranty are themselves guaranteed for 6 (six) months from the day of reception of the repaired product.

Non-defective products

Moretti will notify the customer if concludes that the product is not defective after reception and review. The product will be sent back to the customer at his cost.

18. SPARE PARTS

Moretti original spare parts are guaranteed for 6 (six) months from the day of delivery.

19. NON-LIABILITY CLAUSE

Except as expressly specified in this warranty and within the limits of the law, Moretti makes no declaration, warranty or condition, express or implicit, including any eventual future declarations, warranties or sale conditions, suitability for a given purpose, non violation and non interference. Moretti does not guarantee that the use of Moretti product will be uninterrupted or problem-free. The duration of any implicit warranties imposed by legislations is limited to the warranty period, within the limits of the law. Certain states and countries do not permit limitations of the duration of an implicit guarantee or the exclusion/limitation of accidental or indirect damages in relation to consumer products. In said states and countries, certain exclusions and limitations of this warranty may not be applied to the user. This warranty can be upgraded without notification.

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