GR EN

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INTENDED USE:

These wheelchairs have been designed for physically or mentally handicapped people with limited mobility only for transportation.

0811313

0811312

SPECIFICATIONS

FRAME	Aluminum
COLOR	Black
SEAT	Reinforced Fabric
SIDEBBOARD	Steady
FOOT SUPPORT	Adjustable Length and Tilt
REAR WHEELS	12" Compact
FRONT WHEELS	8" x 2" Compact (200x50mm)
BRAKES	On rear wheels

DIMENSIONS

0811312 0811313	32cm	24-27cm	28cm	38-56cm	66cm	80cm	100-118cm	53cm	12"8"	60kg	16.9kg

ASSEMBLY

Please refer on **ANNEX A**

SYMBOLS

Please refer on **ANNEX B**

MAINTENANCE

Periodic review of the procedures below will help to maintain the life and effectiveness of your wheelchair.

WEEKLY

- Check the wheel pressure by following the instructions on the wheels. If there are pneumatic wheels on the wheelchair.
- Check that the central bolt of the transverse arm is tight and in good condition.
- Clean the fabric with soapy water or sufficient products.
- Clean the metal parts of the chair with a dry cloth or with special products for chrome surfaces or painted surfaces.
- Check the wheel bolts and adjust if necessary.
- Check for damage to front and rear wheels.

QUARTERLY

- Check and adjust if necessary: wheel forks, the wheel axels and the hand-rim of the rear wheels. Grease or oil all components that are subject to friction or impending wear.
- Make sure that all nuts and bolts are tight, but not too tight.
- General revision of the chair's status and repair to whatever is necessary.

ANNUALLY

- We recommend a full review once a year to a qualified service provider.
- For the safety of the patient, we recommend that the wheelchair will be checked by the manufacturer every 2 years. In case of repair, use only authentic spare parts.

CLEANING

In addition to maintenance and inspection, both the frame and the upholstery can be thoroughly cleaned.

- To maintain its appearance, the fabric must be cleaned regularly.
- Dirt can be removed with a sponge and a mild detergent with warm water, sweeping in the direction of texture.
- Rinse with clean water before letting it dry.
- DO NOT leave the wheelchair wet.
- Do not place the fabric in a tumble dryer. If the dirt is deep, the surface should be cleaned with a soft brush.
- The frame should be cleaned regularly with a mild detergent.
- The wheelchairs with leather seats must be cleaned with dry cloth.
- Clean the wheels with warm water and mild soap. Pay attention in drying the wheels. It is dangerous to use the wheelchair with wet wheels.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

IMPORTANT!

The construction of this wheelchair is designed to provide maximum safety. However, users may be at risk by using their wheelchairs unsuitably. For your own safety, the following rules must be observed. Nonspecific or incorrect adjustment work can increase the risk of accidents.

- Always wear light-colored clothes when moving in dark places so that you can be reached more easily from others.
- Be careful during your first course in this wheelchair.
- Meet your wheelchair.
- Make use of your wheelchair on level ground as well as uphill and downhill.
- Do not attempt to reach objects if you need to lift them from the floor reaching between your knees.
- Do not attempt to reach objects if you need to move on to the seat.
- Do not use on roads or motorways.
- Do not attempt to climb or descend with water, ice or oil membrane.
- Do not use unauthorized parts, accessories, or adapters other than those approved, by the manufacturer.
- Do not stand in the wheelchair.
- Do not use the sideboard as a platform when entering or exiting the wheelchair.

SITTING DOWN TO THE WHEELCHAIR:

1. Position the wheelchair as close as possible to you.
2. Engage wheel locks.
3. Lean on the frame with arms and slowly go down towards the seat.

STANDING UP FROM THE WHEELCHAIR:

1. Position the wheelchair as close as possible alongside the seat to which you are transferring.
2. Engage wheel locks.
3. Must rely on both handles equally. Supporting only one hand (and unattended) can cause injury.
4. Shift body weight into the seat that you are going to transfer.

OTHER INSTRUCTIONS:

1. Always use lifts and ramps. If these are not available, you can reach your destination with the help of two escorts. They must lift the wheelchair by holding stable parts of the frame. A wheelchair should never be lifted while a passenger is seated. It should be pushed only.
2. Avoid stairs or obstacles, otherwise ask help from your attendant:
3. This wheelchair must be used solely to carry a person at a time on the seat.

4. Users should not sit in wheelchair while are using any means of transport.
5. The wheelchairs have a brake on the back wheels, which should be secured when the wheelchair is not moving.
6. DO NOT push on the footrests to sit or stand up, DANGER OF OVERTHROW. The footrests are used by the user when He or She is seated.
7. DO NOT touch moving parts while moving DANGER OF INJURY

WARNING

- If you don't read this user manual, it is preferable not to use this product or another available part.
If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the wheelchair by any removable parts.
- Sitting down in and standing up from the wheelchair can be performed independently but requires strength and mobility on the upper body

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels or brakes.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

WARRANTY CLAIM FORM

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE::	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
ADDRESS:			
PHONE:			
			SIGN AND SIGNATURE

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With simple pressure
on the seat support,
the wheelchair
grows. The typical
click sound is heard.

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When the wheelchair is
insured. Looks as
follows.

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Then place the seat in
the corresponding coats
of the wheelchair.

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Place the red seat holders in the correct position on both sides.

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By pressing the two
keys at the same time,
the push arm can
come to the correct
position.

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Place the backrest
extension cover after
passing the zone from
the corresponding
positions. The plastic
must also pass from
the metal part.

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ADJUSTMENT OF THE BACKREST HEIGHT, BACKREST TILT AND SEAT WIDTH

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Remove the backrest cover and seat cushion of the wheelchair. The seat cushion can be removed by opening the buttons as shown in the picture.

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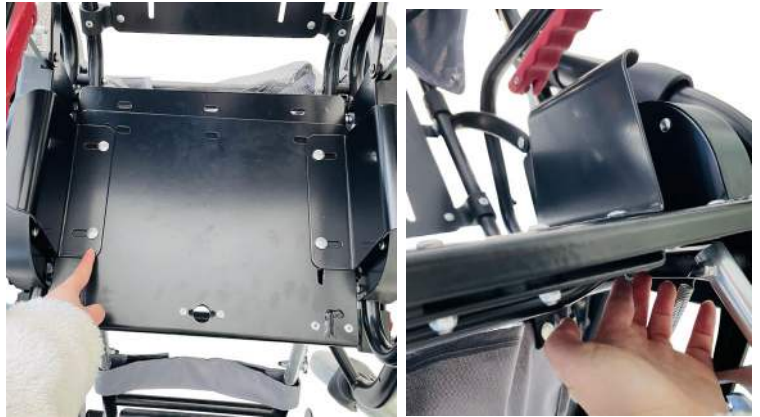
After relaxing the nuts, we have the ability to adjust the height of the headrest to the suitable position.

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Loosen the four screws on the seat and adjust the width to the suitable position.

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Reinstall the seat cushion and backrest cover.

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Relaxing the tilt regulator, we can put the seat to the desired position.

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Add the reclining shade by pressing the adapters to the frame.

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Add the footrests to this position. The tilt is adjusted by the selector.

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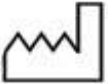
Description of symbols that have been printed on the label, Master Carton and/or User Manual

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